- EN 1. Arm Cuff
- 2. LCD Display
- 3. Air Tube and Connector
- 4. ON/OFF/START key
- 5. Memory Key
- 6. Battery Cover
- 7. Data Link Socket
- 8. AC Adaptor Jack

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EN English

Introduction
Blood pressure measurements determined with CF175f are equivalent to those obtained by a trained observer using cuff/75f auscultation method, within the limits prescribed by the American National Electronic or Automated Sphygmomanometers. This unit is to be used by adult consumers in a home environment. Do not use this device on infants or neonates. CF175f is protected against manufacturing defects by an established International Warranty Program. For warranty information, you can contact the manufacturer, Rossmax International Ltd.

Attention: Consult the accompanying documents. Please read this manual carefully before use. For specific information on your blood pressure, contact your physician. Please see to this manual.

- Before measurement, it's suggested that you sit quietly for at least 5 minutes as measurement taken during a relaxed state will have greater accuracy. You should be relaxed, not physically exerted while taking a measurement.
- Do not take measurements if you are under stress or tension.
- During measurement, do not talk or move your arm or hand muscles.
- Take your blood pressure at normal body temperature. If you are feeling cold or warm, wait a while before taking a measurement.
- If the monitor is stored at very low temperature (near freezing), have it placed at a warm location for at least one hour before using it.
- Wait 5 minutes before taking the next measurement.

Real Fuzzy Measuring Technique
This unit uses the oscillometric method to detect your blood pressure. Before the cuffs starts inflating, the device will establish a baseline cuff pressure equivalent to the air pressure. This unit will determine the appropriate inflation level based on pressure oscillations, followed by cuff deflation.

During the deflation, the device will detect the amplitude and slope of the pressure oscillations and thereby determine for you the systolic blood pressure, diastolic blood pressure, and pulse.

Preliminary Remarks
This Blood Pressure Monitor complies with the European regulations and bears the CE mark "CE 0120". The quality of the device has been verified and conforms to the provisions of the EC council directive 93/42/EEC (Medical Device Directive), Annex I, 1995/42/EEC harmonized and applied harmonized standards.

EN 1060-1:1995/42: 2009 Non-invasive sphygmomanometers - Part 1 - General requirements for a blood pressure measuring system.
EN 1060-2:1997/42: 2009 Non-invasive sphygmomanometers - Part 3 - Supplemental requirements for electro-mechanical blood pressure measuring systems

EN 1060-4: 2004 Non-invasive sphygmomanometers - Part 4: Test Procedures used to determine the overall system accuracy of automated non-asthmatic blood pressure monitoring devices.
This blood pressure monitor was designed for long service time. Ensure consistent accuracy, it's recommended that all digital blood pressure monitors be re-calibrated. This monitor (under normal usage with approx. 3 measurements a day) does not require re-calibration for 2 years. Once the unit should be re-calibrated the device will display CR. This unit should also be re-calibrated if the monitor sustains damage due to blunt force (such as dropping the device) or cold temperatures (such as leaving the device in a humid/dry changes. When CR appears, simply turn to your nearest dealer for re-calibration service.

Recalling Values from Memory
1. To recall stored pressure readings from memory, simply press the Memory key, the first reading displayed is the average of the last 3 measurements stored in memory, and then the last set of memorized readings will be displayed.

2. Another press of the Memory key will recall the previous set of readings. All readings stored in memory will be displayed with its sequence number. Memory will store in memory an assigned memory sequence number.

Clearing Values from Memory
Press and hold the Memory key for approximately 5 seconds, then the data in the memory zone can be erased automatically.

Troubleshooting
If any abnormality will arise during use, please check the following points.
1. Check Monitor Power Switch.
2. No display when the ON/OFF/START key is pressed.
3. Error message is displayed.

Display Explanations
E1 / Measurement Error: Make sure the L-Plug is securely connected to the air socket and measure again quietly. Wrap the cuff correctly and keep your arms steady during measurement. If the error keeps occurring, return the device to your local distributor or service center.
E2 / Pressure Exceeding 300 mmHg: Switch the unit off and measure again quietly. If the error keeps occurring, return the device to your local distributor or service center.
E3 / Data Error: Remove the batteries, wait for 60 seconds and reload. If the error keeps occurring, return the device to your local distributor or service center.
Er / Exceeding Measurement Range: Measure again quietly. If the error keeps occurring, return the device to your local distributor or service center.

Cautionary Notes
1. The unit contains high-precision assemblies. Therefore, avoid extreme temperatures, avoid direct sunlight. Avoid dropping or strongly shaking the main unit, and protect it from dust.
2. Clean the blood pressure monitor body and the cuff carefully with a slightly damp, soft cloth. Do not press. Do not wash the cuff or use chemical cleaner. Never use thinner, alcohol or petrol (gasoline) to clean the device.
3. Leaky batteries can damage the unit. Remove the batteries when the unit is not used for a long time.
4. The unit should not be operated by children so as to avoid hazardous situations.
5. If the unit is stored near freezing, allow it to acclimate at room temperature before use.
6. This unit is not field serviceable. You should not use any tool to open the device nor should you attempt to adjust anything inside the device. If you have any problems, please contact the store or the doctor from whom you purchased this unit or please contact Rossmax International Ltd.
7. As a common issue for all blood pressure monitors using the oscillometric measurement function, the device may have difficulty in determining the proper blood pressure for users diagnosed with common arrhythmias (atrial or ventricular premature beats or atrial fibrillation), diabetes, poor circulation of blood, kidney problems, or for users suffered from stroke, or for unconscious users.

Note: It is strongly recommended that you consult your physician if the IHB icon appears.
Using the AC Adaptor (Optional)
1. Connect the AC adaptor with the AC adaptor jack on the right side of the device.
2. Plug the AC adaptor into the socket. (AC adaptors with required voltage and current indicated near the AC adaptor jack).
Caution:
1. Batteries are hazardous when operating with the AC mode for a long period of time. Leaving the batteries in the compartment for a long time may cause leakage, which may lead to damage of the unit.
2. No batteries are needed when operating with the AC mode.
3. AC adaptors are optional. Please contact the distributor for the compatible AC adaptors.
4. Use only the authorized AC Adaptor with this blood pressure monitor. Information for the authorized AC adaptor, please refer to APPENDIX 1.

Installing Batteries
1. Remove the battery cover in the direction of the arrow to open the battery compartment.
2. Install or replace 4 "AAA" sized batteries in the battery compartment according to the indications inside the compartment.
3. Replace the battery cover by clicking in the bottom hooks first, then pushing the top end of the battery cover.
4. Replace the batteries in pairs. Remove batteries when unit is not in use for extended periods of time.
Caution:
1. Batteries are hazardous when operating with the AC mode for a long period of time. Leaving the batteries in the compartment for a long time may cause leakage, which may lead to damage of the unit.
2. No batteries are needed when operating with the AC mode.
3. AC adaptors are optional. Please contact the distributor for the compatible AC adaptors.
4. Use only the authorized AC Adaptor with this blood pressure monitor. Information for the authorized AC adaptor, please refer to APPENDIX 1.

Specifications

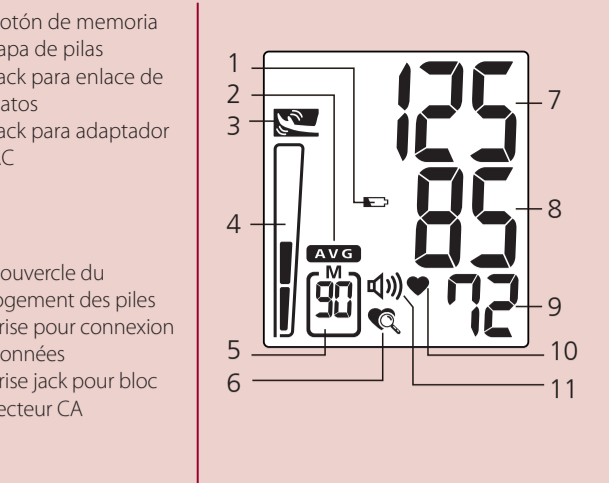
Measurement Method: Oscillometric
Measurement Range: Systolic: 30~260 mmHg; Pulse: 40~199 beats/minute
Pressure Sensor: Mini conductor
Accuracy: Pressure ± 3mmHg; Pulse ± 5% of reading
Inflation: Pump Driven
Deflation: Automatic Air Release Valve
Memory capacity: 90 memories
Auto shut-off: 10 minutes after last key operation
Operation Environment: 10°C~40°C (50°F~104°F); 10%~95% RH; 700~1060 hPa
Storage and Transportation: -10°C~60°C (14°F~140°F); 10%~95% RH; 700~1060 hPa
Environment: 0~40°C (32~104°F)

DC Power Source
DC: 6V four AAA batteries
AC Power Source: DC: 6V, 600mA (Plug size: outer) is Ø40, inner+ is Ø17

Dimensions
Cuff: 124(L) x 85(W) x 85(H) mm (C/W x H x D)
Main Unit: 355(L) x 70(W) x 35(H) mm (C/W x H x D)
Arm circumference: Adult: 22.4~34.0 cm (Ø9.4~13.7")
Limited Users: Adult Users

Type B: Device and cuff are designed to provide special protection against electrical shocks.
Type A: Device and cuff are designed to provide special protection against electrical shocks.
IP Classification: IP21: Protection against harmful ingress of water
Finger approx. 2 cm above the top of the middle of the inside of your left arm. Identify where the pulse can be felt strongest. This is your main artery.

*Specifications are subject to change without notice.
†When the cuff is inflated, the device will detect the amplitude and slope of the pressure oscillations and thereby determine for you the systolic blood pressure, diastolic blood pressure, and pulse.

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